

TENA Comfort Fitting Guide



How to fit TENA Comfort



Fold pad in half lengthways before opening out to fit.



The front of the pad is the smaller area. Place larger area at back.



Pull pants to mid-thigh then turn waistband down to knees.



Pass pad from front to back.



The wetness indicators are the wavy blue lines across the pad.



Pull pad up into position.



Ease leg elastics into groin area.

TENA Comfort - Product Codes

Product	Article Number	Colour Code	Total Absorption Range*	Total Packaging
TENA Comfort Normal	752701	Peach	1000 – 1250mls	126 (3 x 42)
TENA Comfort Plus	752851	Blue	1500 – 1800mls	92 (2 x 46)
TENA Comfort Extra	752951	Yellow	1800 – 2100mls	80 (2 x 40)
TENA Comfort Super	758051	Green	2100 – 2500mls	60 (2 x 30)
TENA Comfort Maxi	759056	Lilac	2900 – 3300mls	56 (2 x 28)

^{*}ISO 11948-1 (Whole Product Testing)

TENA Comfort - NHS Supply Chain Codes

Product	NHS Supply Chain Cat. Code	Colour Code	Total Absorption Range*	Total Packaging
TENA Comfort Normal	CFP1247	Peach	1000 – 1250mls	126 (3 x 42)
TENA Comfort Plus	CFQ800	Blue	1500 – 1800mls	92 (2 x 46)
TENA Comfort Extra	CFQ801	Yellow	1800 – 2100mls	80 (2 × 40)
TENA Comfort Super	CFP1248	Green	2100 – 2500mls	60 (2 × 30)
TENA Comfort Maxi	CFQ802	Lilac	2900 – 3300mls	56 (2 x 28)

^{*}ISO 11948-1 (Whole Product Testing)

TENA *Comfort* is available from the following sources:

TENA CUSTOMER SERVICES –
DIRECT 08456 770679

TENA CUSTOMER SERVICES – HOME DELIVERY 08456 770680 (Home Delivery accounts)



c/o Life Because our products make life easier for you and for millions of people around the world. Because our resources and the way we work are natural parts of the global lifecycle. And because we care.

www.TENA.co.uk