TENA Fix Premium

Designed to meet your needs

To be used in conjunction with body-worn pads, these reusable stretch pants come in six sizes. They have been developed to last longer and to hold both shape and elasticity even with frequent laundering.

TENA Fix Premium has no side seams. With no pressure point, wearers are assured comfort which helps avoid skin chafing and irritation. TENA Fix Premium also has a ribbed waistband which offers a stable and secure fixation of TENA Comfort products.

TENA Fix Premium Extra Small Up to 60cm	Small 50-70cm	Medium 65-90cm	Large 85-110		XXL 140-180cm
Product		rticle ımber	Colour Code	Waist Size	Total Packaging
TENA Fix Premium Extra Small 75		4033	Pink	Up to 60cm	20 x 5
TENA Fix Premium Small		4023	Yellow	50 – 70cm	20 x 5
TENA Fix Premium Medium		4024	Blue	65 – 90cm	20 x 5
TENA Fix Premium Large		4025	Brown	85 – 110cm	20 x 5
TENA Fix Premium Extra Large		4026	Green	100 – 150cm	20 x 5
TENA Fix Premium Extra Extra Large		4027	Orange	140 – 180cm	20 x 5



For the User: The pads are held securely and discreetly in place. The seamless fixation helps avoid skin chafing and irritation.

For the Carer: Six sizes to suit individual requirements with colour coding for easy identification.

For the Purchaser: Product softness, strength and durability mean extended use and greater economy.

Feature	Benefit

Ribbed Waistband	Using TENA Fix Premium, you experience a stable and secure fixation of TENA Comfort
	products – for comfort, performance and peace of mind.
Seamless comfort	TENA Fix Premium has no side seams. With no pressure points, wearers are assured comfort
	which helps avoid skin chafing and irritation.
Just like underwear	TENA Fix Premium is made from a close-mesh, 96% polyester, 4% elasthan material –
	that feels as comfortable and familiar as normal underwear.
Identify each wearer	To help you manage the products there is a space on the waistband to write the name of
	the wearer or room number using a waterproof marker.
Resistant to washing/drying	The design and material ensure you can now wash a pair of fixation pants up to 50 times
	at 60°C. You can also tumble dry them.
Easy to select	Colour-coded according to size for easy and accurate selection.



TENA Comfort using TENA Fix Fitting Guide - Standing & Lying



Fold the pad in half lengthways before opening it out to secure a bowl shape. The front of the pad is the smaller area. Place larger area at the back.



thigh then turn waistband The wetness indicators down to the knees. Pass pad from front to back.



Pull fixation pants to mid- Pull pad up into position. are the blue lines across the pad.



Ease leg elastics into the groin area for secure fixation and leakage security.



Fan-out the product to the sides (front and back) to ensure a close-body and comfortable fit.



Fold the pad in half lengthways before opening it out to fit to secure a bowl shape. The front of the pad is the smaller area. Place larger area at the back.



Pass pad from front to back. The wetness indicators are the blue lines across the pad.



Pull fixation pants up over Ease leg elastics into the pad.



the groin area for secure fixation and leakage security.



Fan-out the product to the sides (front and back) to ensure a close-body and comfortable fit.

TENA Fix Premium recommended domestic washing procedure



Dry Cleanable



Do not use Chlorine Bleach



Cool iron



Can be machine washed



Do not dry on a radiator

Hygiene Support



Email: hcmarketing@sca.com www.TENA.co.uk/professionals



TENA Nursing Support