


TENA Men

Engineered especially for men

TENA Men, available in three absorbency levels, is especially designed to fit the male form- ensuring maximum security and discretion. The cup shaped pad follows the contours of the body to give a snug and secure fit. The adhesive strip secures the pad in position within normal underwear. The soft material is gentle to the skin and completely noise free. The unique Odour Control™ inhibits the development of unwanted smells for a more dignified way of life.

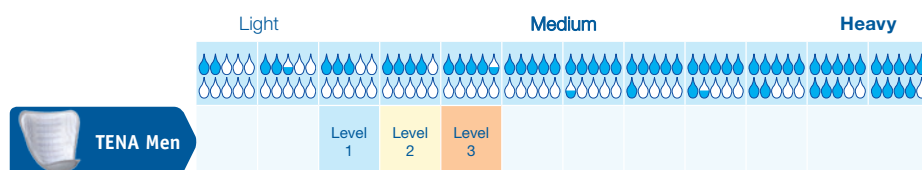

Product	Article Number	Working Absorbency	Total Packaging
TENA Men Level 1	750651	150mls	144 (6 x 24)
TENA Men Level 2	750776	200mls	120 (6 x 20)
TENA Men Level 3	750830	300mls	96 (6 x 16)



For the User: A soft, quiet backing material with adhesive release paper provides a discreet and comfortable fit. Single-wrapped for ease of transport and disposal.

For the Carer: Easy to fit and change.

For the Purchaser: An effective low-cost solution.


Feature

- Fast absorption, dry surface..... Reduces skin irritation
- Adhesive strip holds the pad in place in the underwear Fixing the pad to snug fitting underwear minimises the risk of leakage
- Unique Odour Control Urine is locked away in the pad, preventing the development of unwanted odour for discretion and comfort
- Leak proof breathable textile-like backing The breathable backing improves comfort and reduces skin irritation to prevent staining
- Anatomically shaped pad The pad is designed to fit the male anatomy in order to provide maximum comfort and leakage security and reduce odours
- Masculine design For discretion
- Single-wrapped products For added discretion and hygiene


Benefit



TENA Men Fitting Guide

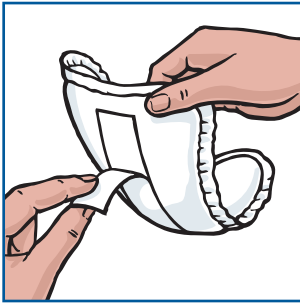

Open the TENA Men packaging and unfold the pad, removing the tab to expose the adhesive strip.

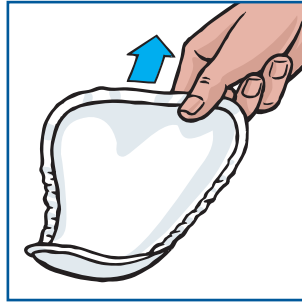

Make sure that the wider part of the pad is positioned upwards.



Place the pad in snug-fitting underwear so that the adhesive strip attaches to the inside front.



Do not use with loose-fitting underwear or boxer shorts.

TENA Men dos and don'ts

- ✓ Assess user for correct absorbency
- ✓ Fold pad lengthways to create bowl shape
- ✓ Fit with wide end of pad pointing upward
- ✓ Ensure a close fit is achieved
- ✗ Never place one pad inside another to increase absorbency
- ✗ Never use talc, as it reduces the absorbency of the product
- ✗ When creams are prescribed, used sparingly to ensure the product works effectively

People who bought TENA Men also bought...

TENA Pants Normal

TENA Pants Normal provides outstanding security through Feeldry™ technology, which rapidly absorbs even large quantities of urine into the product core and quickly away from the skin, keeping the surface exceptionally dry. The range also incorporates Body-Close Fit™, resulting in increased discretion and an improved sense of dignity. TENA Pants offer outstanding dryness, comfort and healthier skin for the user.



TENA Shampoo Cap

The new TENA Shampoo Cap is pre-moistened with a non-rinse hair washing formula. Ideal in situations where normal hair washing is challenging or not possible.



TENA Wash Cream

This 3-in-1 wash cream cleanses, restores and protects even the most delicate and fragile skin.

SCA Hygiene Support



Email: hcmarketing@sca.com
www.TENA.co.uk/professionals



TENA Nursing Support

