

TENA *Pants*

NOW WITH SUBTLE WETNESS INDICATOR
NEW PRODUCTS

The modern solution that's just like underwear

A soft, comfortable, disposable absorbent product that can be handled just like normal underwear, encouraging greater independence. TENA *Pants* Discreet offers superb security and discretion. They are thin, soft and stretch to the shape of the body. TENA *Pants* provides outstanding security through FeelDry™ technology, which rapidly absorbs even large quantities of urine into the product core and quickly away from the skin, keeping the surface exceptionally dry.

The product range also incorporates Body-Close Fit™, resulting in increased discretion and an improved sense of dignity. TENA *Pants* offer outstanding dryness, comfort and healthier skin for the user. Now featuring a subtle wetness indicator to facilitate effective product use*. Available in five absorbency levels. Latex free.

	Light				Medium				Heavy			
TENA												
Pants												

Product	Article Number	Hip Size	Working Absorbency	Total Packaging
TENA <i>Pants</i> Discreet Medium	790300	75 – 100cm	300mls	48 (4 x 12)
TENA <i>Pants</i> Discreet Large	793300	95 – 125cm	300mls	40 (4 x 10)
TENA <i>Pants</i> Normal Medium	790118	80 – 110cm	400mls	72 (4 x 18)
TENA <i>Pants</i> Normal Large	790218	100 – 135cm	400mls	72 (4 x 18)
TENA <i>Pants</i> Plus Extra Small	790901	50 – 70cm	700mls	44 (4 x 11)
TENA <i>Pants</i> Plus Small	791002	65 – 85cm	600mls	56 (4 x 14)
TENA <i>Pants</i> Plus Medium	791102	80 – 110cm	600mls	56 (4 x 14)
TENA <i>Pants</i> Plus Large	791202	100 – 135cm	600mls	56 (4 x 14)
TENA <i>Pants</i> Plus Extra Large	791302	120 – 160cm	600mls	48 (4 x 12)
TENA <i>Pants</i> Super Small	791060	65 – 85cm	800mls	48 (4 x 12)
TENA <i>Pants</i> Super Medium	791160	80 – 110cm	800mls	48 (4 x 12)
TENA <i>Pants</i> Super Large	791260	100 – 135cm	800mls	48 (4 x 12)
TENA <i>Pants</i> Super Extra Large	791360	120 – 160cm	800mls	48 (4 x 12) NEW
TENA <i>Pants</i> Maxi Medium	791170	80 – 110cm	1050mls	40 (4 x 10) NEW
TENA <i>Pants</i> Maxi Large	791270	100 – 135cm	1050mls	40 (4 x 10) NEW

For the User: Can easily be pulled up and down like regular underwear which encourages more self-care and helps to promote continence. A very discreet and more dignified solution.

For the Carer: Adhesive strip at the back is used to wrap the used product for easy disposal. Tear-away side seams make removal easier. Fewer products are required, reducing the workload associated with fitting and changing.

For the Purchaser: Reduced product usage facilitated by newly added wetness indicators and continence promotion saves time and money.

Feature

- Feeldry™ Layer
- Subtle wetness indicator
- Body-Close Fit™
- 100% breathable™
- Anti Leakage™
- Adhesive tape
- Tear away seams
- Double cuffs
- Elastic Waist Band


Benefit

- For quick absorption of liquid for maximum leakage security; better dryness for improved skin care.
- For increased confidence for the individual and the carer.*
- Improved fit for greater security and discretion.
- Providing extra comfort and helps skin stay healthy.
- To provide comfort and snug fit.
- To dispose discreetly.
- For easy removal.
- To give maximum protection against leakage.
- For better fit and better comfort.


*With exception of TENA *Pants* Discreet.



TENA *Pants* Fitting Guide – Standing

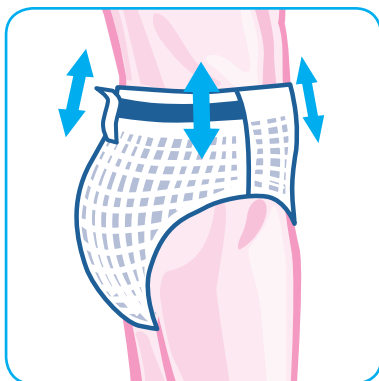

Pull up just like normal underwear.

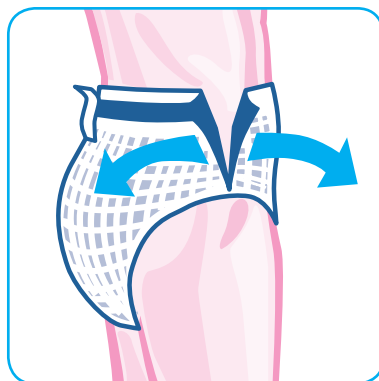

Tear at the sides for easy removal.

TENA *Pants* do's and don'ts

- ✓ Measure around the hips for correct size
- ✓ Assess for correct absorbency
- ✓ Pull pad all way up
- ✓ Ensure a close fit is achieved
- ✓ Tear side-seam to remove pad
- ✓ Use tape to wrap pad for disposal
- ✗ Never place one pad inside another to increase absorbency
- ✗ Never use talc, as it reduces the absorbency of the product
- ✗ When creams are prescribed, use sparingly to ensure the product works effectively

People who bought TENA *Pants* also bought...



TENA *Bib*

These handy, disposable serviette bibs come with integral front pockets and back ties for dignified and hygienic protection at meal times.



TENA *Wipes*

TENA Wet Wipes are pre-moistened with a 3-in-1 formula of surfactant free emulsions to cleanse, restore and protect skin with just a single wipe.



TENA *Wash Cream*

This 3-in-1 wash cream cleanses, restores and protects even the most delicate and fragile skin.

SCA Hygiene Support



TENA Advice Line: 0845 308080 30



TENA Education Academy



www.TENA.co.uk



TENA Nursing Support