


TENA Slip

A traditional all-in-one.

For those who require or prefer an All-in-one product, TENA Slip offers increased absorption and leakage security in three levels and sizes to meet individual requirements. TENA Slip has a FeelDry™ layer that provides outstanding dryness and leakage security. Latex free.

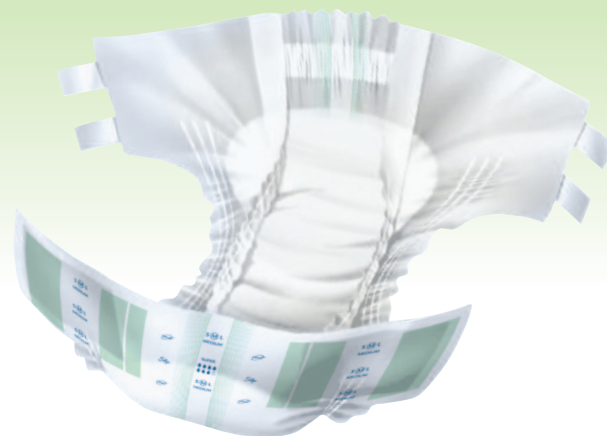

	Light				Medium				Heavy		
TENA											
Slip											

Product	Article Number	Hip Size	Total Absorption Range*	Total Packaging
TENA Slip Extra Small	713028	40 – 60cm	1300 – 1700mls	84 (3 x 28)
TENA Slip Plus Small	713128	50 – 80cm	1700 – 2100mls	84 (3 x 28)
TENA Slip Plus Medium	713228	70 – 110cm	2100 – 2500mls	84 (3 x 28)
TENA Slip Plus Large	713328	100 – 150cm	2100 – 2500mls	84 (3 x 28)
TENA Slip Super Small	714128	50 – 80cm	1700 – 2100mls	84 (3 x 28)
TENA Slip Super Medium	714228	70 – 110cm	2500 – 2900mls	84 (3 x 28)
TENA Slip Super Large	714328	100 – 150cm	2500 – 2900mls	84 (3 x 28)
TENA Slip Maxi Small	715121	50 – 80cm	2100 – 2500mls	63 (3 x 21)
TENA Slip Maxi Medium	715221	70 – 110cm	2900 – 3300mls	63 (3 x 21)
TENA Slip Maxi Large	715321	100 – 150cm	3700 – 4100mls	63 (3 x 21)

For the User: Curved leg elastics and security barriers provide increased comfort and greater protection against urinary and faecal incontinence.

For the Carer: The unique Superfit™ Band allows the fixation tapes to be resealed and repositioned to ensure a snug fit. Colour-coded for correct product selection. Excellent leakage protection reduces frequency of pad changes.

For the Purchaser: Reduced product usage and continence promotion saves time and money.

Feature

- Feeldry™ Layer
- Easy Fit™ Re-fastenable tapes
- Tape Landing Zone™
- Odour Neutralizer™
- Body Fitting™
- Wetness Indicator
- FlexiFit™ Waistband
- Curved leg elastics
- Colour Coded

Benefit

- For quick absorption of liquid for maximum leakage security; better dryness for improved skin care.
- Can be re-adjusted as many times to ensure good comfort and fit.
- Dedicated area for tapes to be sealed.
- To reduce any unpleasant odours.
- For better comfort & close body fit.
- To advise when the pad needs changing.
- For better comfort & close body fit.
- For better comfort & close body fit.
- For easy identification.



TENA Slip Fitting Guide – Standing and Lying



TENA Slip do's and don'ts

- ✓ Measure around the hips for correct size
- ✓ Assess for correct absorbency
- ✓ Fold pad lengthways to create bowl shape
- ✓ Fit from front to back
- ✓ Seal bottom tabs pointing upwards and top tabs pointing downwards
- ✓ Ensure a close fit is achieved
- ✗ Never place one pad inside another to increase absorbency
- ✗ Never use talc, as it reduces the absorbency of the product
- ✗ When creams are prescribed, used sparingly to ensure the product works effectively

People who bought TENA Slip also bought...



TENA Bib

These handy, disposable serviette bibs come with integral front pockets and back ties for dignified and hygienic protection at meal times.



TENA Wipes

TENA Wet Wipes are pre-moistened with a 3-in-1 formula of surfactant free emulsions to cleanse, restore and protect skin with just a single wipe.



TENA Wash Cream

This 3-in-1 wash cream cleanses, restores and protects even the most delicate and fragile skin.

SCA Hygiene Support



TENA Advice Line: 0845 308080 30



www.TENA.co.uk



TENA Education Academy



TENA Nursing Support



SCA HYGIENE PRODUCTS UK LIMITED,
Personal Care, Southfields Road
Dunstable, Bedfordshire LU6 3EJ
Tel: 01582 677400
www.TENA.co.uk

c/o Life Because our products make life easier for you and for millions of people around the world. Because our resources and the way we work are natural parts of the global lifecycle. And because we care.