


# TENA Slip Stretch XXL

## Specifically designed for Quality Bariatric Care

TENA Slip Stretch XXL offers a new definition of comfort and complete leakage security for individuals with hip sizes of up to 178cms. Available in one size and absorbency level, TENA Slip Stretch XXL features a soft textile back-sheet and Easy To Apply Reclosable fixation Wide Hooks enabling multiple re-adjustments.

Straightforward for caregivers to achieve the perfect fit, the Breathable Stretch Side panels provide a comfortable fit and gently stretch to follow the body's movements. Also featuring FeelDry™ technology to quickly draw liquid away from skin to keep skin healthy. Available in Super absorbency to provide complete leakage protection.

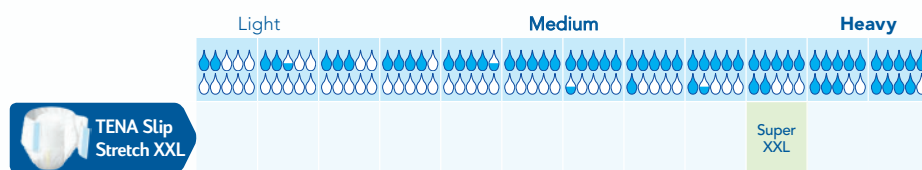

Product	Article Number	Colour Code	Hip Size	Working Absorbency	Total Packaging
TENA Slip Stretch XXL	61490	Green	163 – 178cm	1200mls	64 (2 x 32)



**For the User:** New XXL size product provides increased comfort and superior fit for hip sizes of up to 178cms.

**For the Carer:** Wide Hook side panels are Reclosable and allow the fixation tabs to be resealed and repositioned multiple times.


**For the Purchaser:** High absorption capacity facilitates less frequent pad changes, reducing total costs.

## Feature

## Benefit

- **FeelDry™ Layer** ..... For quick absorption of liquid for maximum leakage security; enhanced dryness for improved skin care
- **Easy To Apply** ..... Reclosable fixation hook and eye fastenings can be readjusted multiple times to ensure good comfort and fit
- **Odour Neutralizer** ..... To reduce any unpleasant smells
- **Breathable Stretch Sides** ..... Gently stretch to follow the body's movements for a comfortable fit
- **Wide Hooks** ..... Fit securely to dedicated fixation area for added confidence
- **Soft Textile Backing** ..... Ensures enhanced comfort and dryness for improved skincare
- **Wetness Indicator** ..... To advise when the pad needs changing
- **Curved leg elastics** ..... For enhanced comfort and close body fit
- **Suitable for hip measurements up to 178cm** ... New product available in our largest ever size


# TENA Slip Stretch XXL Fitting Guide – Lying and Standing

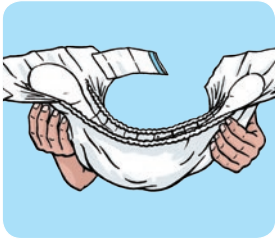


Fully open the pad and fold it lengthways into a bowl shape.



With the individual lying on their side and the pad at the back, position the pad with the fixation tabs in line with the hips.

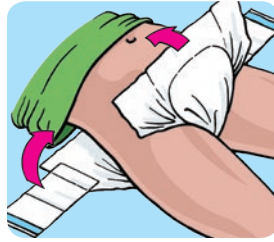


With the individual lying on their back, gently pull the pad between the legs.

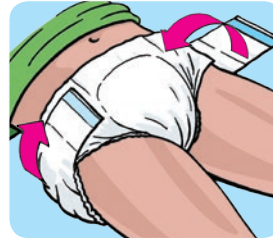


Fit the Wide Hook fixation tabs straight across the hips and fasten securely.




Check for comfort and fit. The pad allows for refastening and adjustment of the hook and eye fixation.

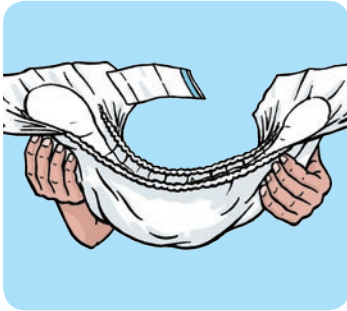

Fully open the pad and fold it lengthways into a bowl shape.

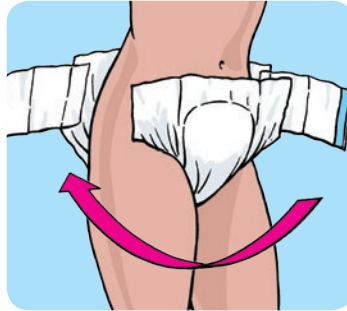

The end of the pad with fixation tabs is the back of the pad. With the individual standing, place the pad between the legs from front to back.

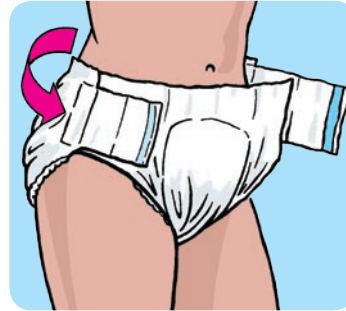

With the individual standing, fit the Wide Hook fixation tabs straight across the hips and fasten securely on both sides.



Check for comfort and fit. The pad allows for refastening and adjustment of the hook and eye fixation.

## TENA Slip Stretch XXL dos and don'ts

- ✓ Measure around the hips for correct size
- ✓ Assess for correct absorbency
- ✓ Fold pad lengthways to create bowl shape
- ✓ Fit from front to back
- ✓ Position fixation tabs pointing straight across
- ✓ Ensure a close fit is achieved
- ✗ Never place one pad inside another to increase absorbency
- ✗ Never use talc, as it reduces the absorbency of the product
- ✗ When creams are prescribed, used sparingly to ensure the product works effectively

## People who bought TENA Slip Stretch XXL also bought...



### TENA Bib

These handy, disposable serviette bibs come with integral front pockets and back ties for dignified and hygienic protection at meal times.



### TENA Shampoo Cap

The new TENA Shampoo Cap is pre-moistened with a non-rinse hair washing formula. Ideal in situations where normal hair washing is challenging or not possible.



### TENA Wash Cream

This 3-in-1 wash cream cleanses, restores and protects even the most delicate and fragile skin.

## SCA Hygiene Support



Email: [hcmarketing@sca.com](mailto:hcmarketing@sca.com)



[www.TENA.co.uk](http://www.TENA.co.uk)



TENA Education Academy



TENA Nursing Support

SCA HYGIENE PRODUCTS UK LIMITED,  
Incontinence Care Europe, Southfields Road,  
Dunstable, Bedfordshire LU6 3EJ  
Tel: 01582 677400  
[www.SCA.com](http://www.SCA.com), [www.TENA.co.uk](http://www.TENA.co.uk)

SCA *Care of life*. Because our products make life easier for you and for millions of people around the world. Because our resources and the way we work are natural parts of the global lifecycle. And because we care.



DI010 1214